

Patch Test User Information:

Please refer to the enclosed panel information sheet which shows the Panel number and the specific allergen placed in each Chamber of each panel.

1. Carefully open the patchTransport.
2. Carefully remove the panel from the patchTransport.
3. The panel number is labeled on the **TOP** of the panel.
4. Please be aware that the Panel Information Sheet shows the placement of panels and allergens as they will appear when applied to the patient. Notice therefore, the **allergens #6-10 are on the left side** of the panel and **allergens #1-5 are on the right side** of the panel.
5. Carefully remove the clear plastic numbered chamber cover.
6. If using: Finn Chamber on Scanpor Open, Finn Chamber AQUA, allergEAZE Non-Woven, or allergEAZE Clear, remove the paper backing to reveal the adhesive/adhesive side.
7. If using Gel Chips on polyester tape, remove the plastic backing to reveal the adhesive side.
8. The Panel number will be visible on the top back side of the Panel for proper orientation.
9. When interpreting the patch test results, refer to the Panel Information Sheet to properly identify the allergens.

****REFRIGERATE ALLERGEN PANELS UPON RECEIPT****

The panel(s) should be applied to the patient within 7 days of receipt
but can be used up to the Discard After date on the RX label.

**Any questions? Please call the pharmacy
at 1-800-344-5341 Option 1.**

Liquid & Volatile Syringe Organizing System (LVSOS)

(Liquid  and  volatile allergens should only be dispensed immediately before the application to the patient.)

1. Open patchTransport to expose Panel 1.
2. Place Panel 1 **LVSOS** card Chamber 1-5 to the right of the Panel.
3. Place Panel 1 **LVSOS** card Chamber 6-10 to the left of the Panel.
4. Remove clear plastic numbered chamber cover to expose chambers. Place near panel for reference.
5. Tear first syringe off the **LVSOS** card and immediately apply the entire contents of the volatile allergen or 1 drop of the liquid allergen to the corresponding Chamber.
6. Dispose of empty syringe properly.
7. **Repeat for each syringe in the LVSOS card.**
8. Dispose of empty card.
9. **Repeat for each Panel.**
10. Panel(s) are now ready to apply to patient's skin.

